

QUICK REFERENCE GUIDE

IV Compatibility Enhancements

Designed to improve usability, provide more comprehensive data, and ensure that you have the latest information at your fingertips for safe and efficient IV medication compatibility checks

Seeing the Chart View vs. List View

The default view is Chart View, but you can toggle to the List View

Default Filters (Drug-Drug & Drug-Solution)

Administrative Method = Y-Site
 Compatibility = All
 Modify filters as needed

Print Feature

Print IV compatibility results easily

Icon Legend

- Compatible – Evidence determined these items are compatible
- Incompatible – Evidence determined these items are incompatible
- Uncertain – Evidence was inconclusive and compatibility was not determined
- No Data – No compatibility data identified
- Variable - Evidence shows the compatibility of the pair is variable

Variable Results


What it means: verify the details before administering the drugs.

What to do: When you see this icon, it's important to:


Look into the specifics to understand the safety of the drugs together, select results

If unsure, consult with the Pharmacy for guidance.


New: Most Searched Drug Pairs

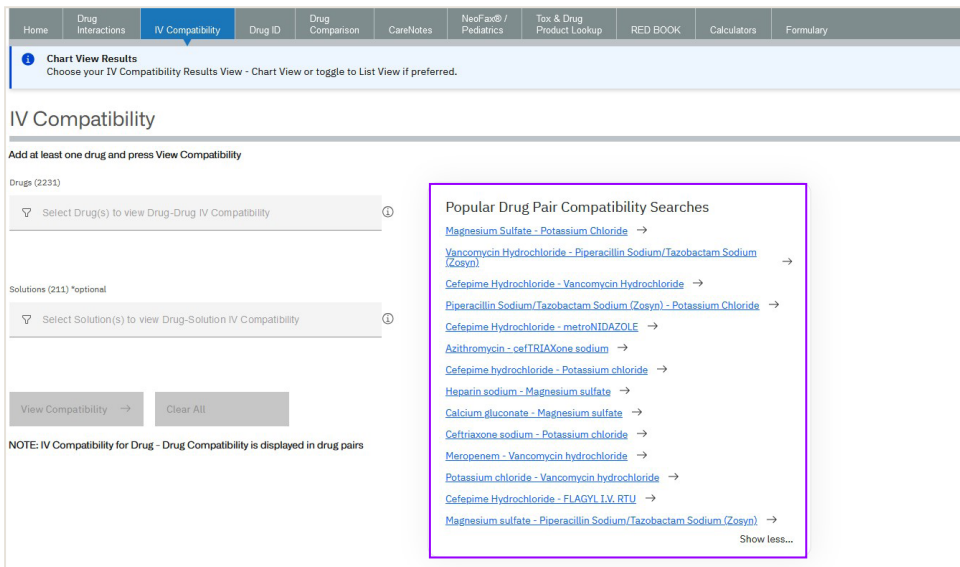

Workflow: IV Compatibility (single) now opens within the IV Compatibility tool and auto-populates drug name from the monograph

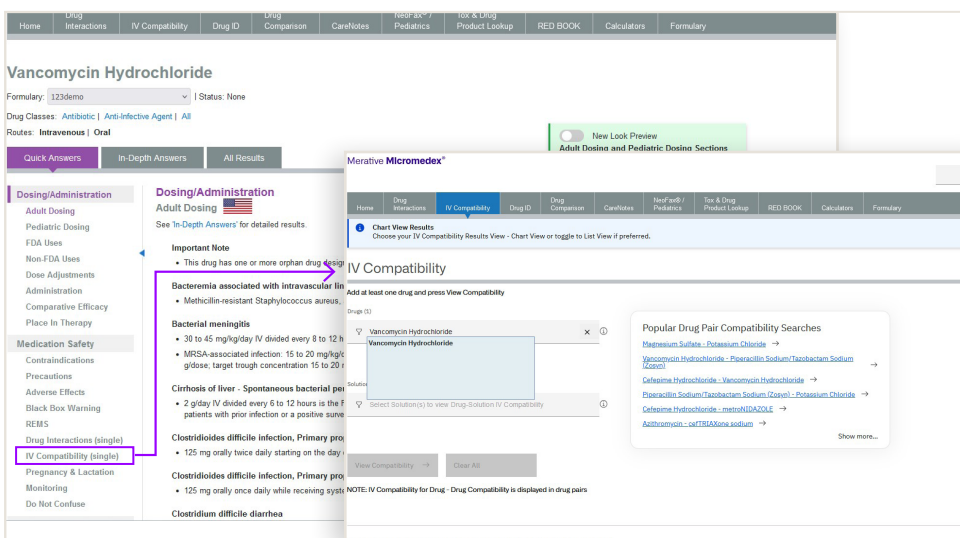

IV Compatibility Screens are optimized for multiple screen sizes ranging from desktop to tablet and mobile phone displays and available on coming soon Mobile Micromedex

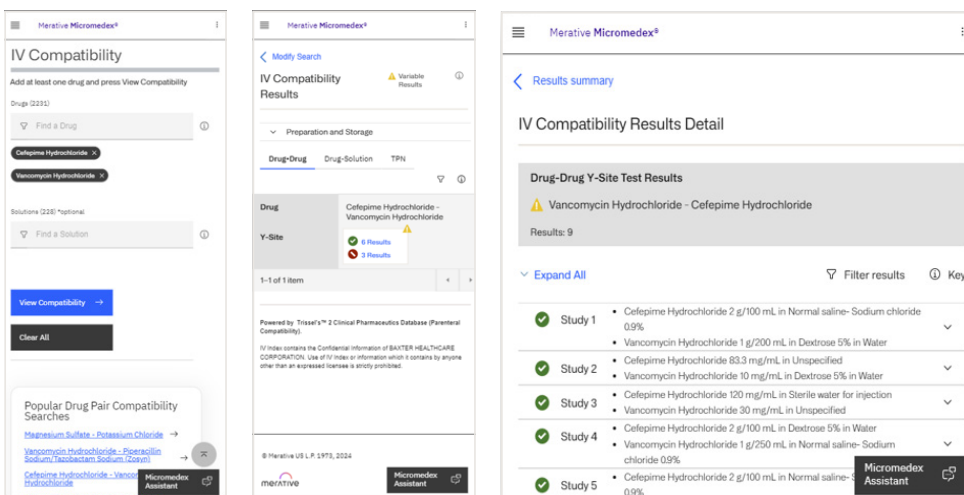

Support & Access: For more information or support, contact your account manager or Micromedex Product Support 24/7 at 1.877.843.6796 (US & Canada) or +1.734.768.1510 (Global)